

Prontosan® Wound Spray

Cleansing and moistening of superficial wounds and burns





The **Prontosan® Wound Spray** supports rapid wound healing by effective cleansing and moistening of superficial wounds and burns.

Application

Thoroughly spray Prontosan® Wound Spray onto the entire wound and the wound site. With this, the risk for infection is reduced and optimal healing conditions are generated. Afterwards, cover the wound with a dressing. Clotted or encrusted dressings can be moistened with Prontosan® Wound Spray in order to be easily removed and to avoid pain during removal.

HANDLING/APPLICATION TIP
Rinse off residues of soaps, ointments, oils, or other wound cleansing substances thoroughly prior to
the use of Prontosan* Wound Spray.

ADVANTAGES

- Prevents wound infection
- Supports wound healing
- Prevents MDRO contamination
- Prevents biofilm formation
- Reduces pain
- Painless application

INDICATIONS

For cleansing, irrigation and moistening of

- Superficial acute and chronic wounds (e.g. skin lesions, bites, lacerations, abrasions, crush wounds, decubitus ulcers, ulcus cruris, diabetic ulcers)
- Superficial burns
 Supports removal of encrusted bandages.

Prontosan® Wound Spray	Size	Pcs/Pack	Article number
Spray bottle	75 ml	20	400565

Composition: Purified Water, Betaine Surfactant, 0.1 % Polyaminopropyl Biguanide (Polihexanide)



B. Braun Medical AG | Infection Control | Seesatz 17 | CH-6204 Sempach Phone +41 58 258 50 00 | Fax +41 58 258 60 00 | info.bbmch@bbraun.com | www.bbraun.com