

PRONTOSAN® DEBRIDEMENT PAD

SOFT MECHANICAL REMOVAL OF SLOUGH AND DEBRIS

B. Braun Wound Care. Right. From the start.

Prontosan® Debridement Pad

Soft mechanical removal of slough and debris

Designed for soft mechanical debridement of slough and debris. The Prontosan® Debridement Pad removes and binds slough and debris and assists with effective wound bed preparation - new granulation tissue is left intact.

FEATURES

- Good cleansing and debridement due to microfiber technology
- Soft debridement, no tissue irritation
- Unique droplet shape to allow debridement of shallow cavities and areas difficult to reach
- Blister packaging to allow safe and aseptic soaking of the pad prior to use
- Produces good results even with scaly and necrotic coatings.

INNOVATIVE SHAPE - DESIGNED FOR SUCCESS




Micro Technology, Maximum Cleansing

The Prontosan® Debridement Pad uses the latest technology in cleansing - microfibres. Microfibres are very small and utilise 'electrostatic forces' to attract even the smallest particles to them.

Each microfibre has a multiple strand structure, allowing for many more particles of slough and debris to be **removed** from the wound bed and **bound** to each microfibre within the pad; not just brushed away as with traditional, much larger, monofilament fibres.

There are **millions of microfibres** in each Prontosan® Debridement Pad for magnified debridement power.

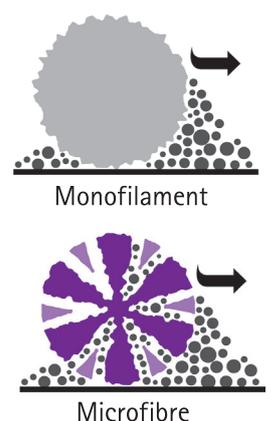

Experts in Wound Cleansing

A new addition to complete the Prontosan® range

BEFORE AND AFTER USING THE PRONTOSAN® DEBRIDEMENT PAD



Before debridement



After a single debridement



3 weeks post debridement

HOW TO USE FOR BEST RESULTS

TIP

- B. Braun branded side faces away from the wound bed
- Prontosan® Solution (as soak or rinse) can be used beforehand

1

Open the packaging, using the integrated tray to moisten (with 15-20 mls Prontosan Solution*), covering the microfibre side of the pad

2


Applying light pressure, use circular or sweeping motions over areas of slough and debris

3

Irrigate (with Prontosan Solution*) to cleanse

4

Apply Prontosan Wound Gel /Gel X to help prevent biofilm formation and apply dressings as appropriate**

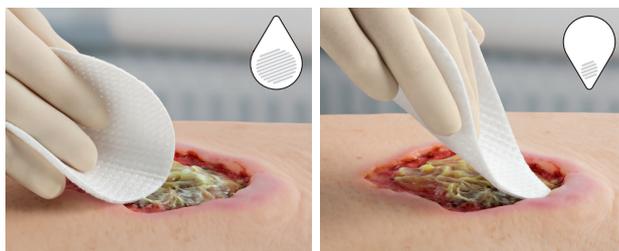

Following debridement, it is advisable to irrigate the wound thoroughly for a second time with Prontosan® Wound Irrigation Solution and to continue with standard treatment.

CLEANSING STRENGTH

One Prontosan® Debridement Pad is able to remove wound debris on wounds with a surface approx. 10 x 15cm (will depend on the amount of wound slough, debris etc).



Before use of Prontosan® Debridement Pad



After use of Prontosan® Debridement Pad

Prontosan® Debridement Pad

Ordering Information

Pcs/Pk	Product Size	Article Number
Pack of 10 Pads	Droplet shape 12.76 x 9.2 cm	3908457
Pack of 3 Pads		3908456



B. Braun Australia Pty Ltd | Bella Vista NSW | Tel. 1800 251 705 | info@bbraun.com | www.bbraun.com.au
B. Braun New Zealand | Customer Care 0800 227 286 | Fax (09) 373 5601 | www.bbraun.co.nz

BAUS OM E0579 03/20 | ©2020 B. Braun Australia