

# Surecan Safety II

## High pressure resistant non-coring safety needle for long term infusion

Surecan Safety II needles consist of an ergonomic wings and transparent, round base system.

The wings and base are attached to each other via the retaining clip located on the base. If the wings dislodge from the base, prior to accessing the port, simply clip them back together at the retaining clip.

There are two breathing holes and a non-absorbent pad for patient comfort.

The silver pad covers a green dot which forms part of the safety mechanism. When the green dot is revealed, the safety mechanism is activated.



### HOW TO ACCESS A PORT




Before inserting the needle, prime the device and disinfect the skin as per hospital protocol.



Hold the needle with the wings and insert into the access port septum at a straight, 90°C angle. The needle should touch the base plate of the access port.



The needle should sit flush to the skin. If there is a gap consider using a shorter needle.

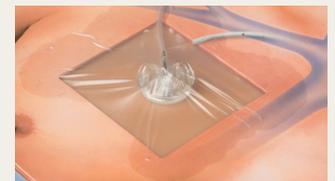

Lay the wings flat onto the access port and cover with a clear dressing.

### HOW TO REMOVE THE NEEDLE



Remove the dressing and clasp the wings together, firmly hold the base of the needle while allowing you to see the silver pad.



Firmly pull the wings upwards in a straight angle whilst holding the base onto the patient.



You will feel/hear the device click into its safety mechanism.



The green dot will appear on the base and the device can safely be removed from the patient.