



Easypump[®] II with Multi-Ad[®] Fluid Transfer Set

Filling the Easypump® II with Multi-Ad® Transfer Set

The convenient solution for filling Easypump® II

Within the following handling instruction, the components from Multi-Ad® Transfer Set and Easypump® II are differentiated by color: Multi-Ad® Transfer Set and Easypump® II.

User benefits

- Convenient filling elastomeric infusion pumps
- Reduces preparation time by avoiding multiple connection and disconnection of the syringe
- Manufactured without Latex and DEHP

1 Close clamp



Close the **clamp of Multi-Ad® Transfer Set**. Fully insert **spike** into outlet of solution container and suspend it from a suitable stand.

When using a flexible container (infusion bag or Ecoflac® plus) close the venting cap at the spike. When using a rigid container (glass bottle) open the venting cap.

2 Connect syringe, remove cap



Remove **protection cap** of the **female luer-lock connector** and connect a syringe. Remove the **protection cap** of the **male luer-lock connector**.

3 Open clamp



Open the **clamp**. To prime the system, withdraw liquid from the container with the syringe. Remove air that came into the syringe during the priming process via the **male luer-lock connector** until a small drop can be seen.

4 Open comfort cap



Close the **clamp of Easypump® II**. Flip open the **comfort cap** and twist off the **closing cone**. Place it on a sterile surface. Connect **Easypump® II** to the **male luer-lock connection** of **Multi-Ad® Transfer Set**.

5 Fill and inject



Fill the syringe completely and inject the diluent into **Easypump® II**.

6 Prime



Prime **Easypump® II**.

For further information, please see the priming instructions for Easypump® II.

7 Fill pump



Fill the syringe again and inject the diluent into the pump. Repeat this step until the required volume (minus 10 ml) is filled into the pump.

8 Inject medication



Disconnect the syringe. Attach a syringe filled with medication to the **female luer lock connection**. Inject medication into **Easypump® II**.

9 Rinse connection part, close clamp



Fill the syringe with approx. 10 ml of the diluent and inject it into **Easypump® II** to rinse the connection part. Close the **clamp of Multi-Ad® Transfer Set**. Disconnect **Easypump® II** from **Multi-Ad® Transfer Set**. Reattach the **closing cone** and the close the **comfort cap**.

Product information

Product	Length	Units per box	Code no. (REF)
Multi-Ad® Transfer Set	109.2 cm	20	513548

