

Prontosan®

## Debridement Pad

Soft Mechanical Removal of Slough  
and Debris



# Prontosan® Debridement Pad

## Soft mechanical removal of slough and debris

Designed for soft mechanical debridement of slough and debris. The Prontosan® Debridement Pad removes and binds slough and debris and assists with effective wound bed preparation – new granulation tissue is left intact.

### Features

- Good cleansing and debridement due to microfiber technology
- Soft debridement, no tissue irritation
- Unique droplet shape to allow debridement of shallow cavities and areas difficult to reach
- Blister packaging to allow safe and aseptic soaking of the pad prior to use
- Produces good results even with scaly and necrotic coatings.

### Innovative shape- Designed for success




### Mirco Technology, Maximum Cleansing

The Prontosan® Debridement Pad uses the latest technology in cleansing – microfibres. Microfibres are very small and utilise 'electrostatic forces' to attract even the smallest particles to them.

Each microfibre has a multiple strand structure, allowing for many more particles of slough and debris to be **removed** from the wound bed and **bound** to each microfibre within the pad; not just brushed away as with traditional, much larger, monofilament fibres.

There are **millions of microfibres** in each Prontosan® Debridement Pad for magnified debridement power.

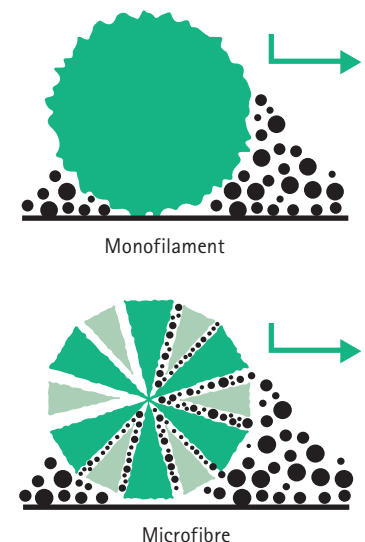

# Experts in Wound Cleansing

A new addition to complete the Prontosan® range

Before and after using the Prontosan® Debridement Pad



Before debridement



After a single debridement



3 weeks post debridement

## How to use for best results

### Tip:

- B. Braun branded side faces away from the wound bed
- Prontosan® Solution (as soak or rinse) can be used beforehand


1. Open the packaging, using the integrated tray to moisten (with 15-20mls Prontosan Solution\*), covering the microfibre side of the pad
2. Applying light pressure, use circular or sweeping motions over areas of slough and debris
3. Irrigate (with Prontosan Solution\*) to cleanse
4. Apply Prontosan Wound Gel/Gel X to prevent biofilm formation and apply dressings as appropriate\*\*

## Cleansing strength

One Prontosan® Debridement pad is able to remove wound debris on wounds with a surface approx. 10 x 15cm (will depend on the amount of wound slough, debris etc).



Following debridement, it is advisable to irrigate the wound thoroughly for a second time with Prontosan® Wound Irrigation Solution and to continue with standard treatment.

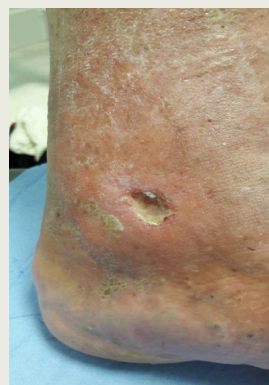

Before use of Prontosan® Debridement Pad



After use of Prontosan® Debridement Pad

# Prontosan® Debridement Pad

## Ordering Information

Pcs/Pk	Product Size	Article Number
Pack of 10 Pads	Droplet shape 12.76 x 9.2 cm	3908457
Pack of 3 Pads		3908456



B. Braun Australia Pty Ltd | Level 2, 4 Burbank Place, Norwest NSW 2153, Australia | Tel.1800 251 705 | [info.au@bbraun.com](mailto:info.au@bbraun.com) | [www.bbraun.com.au](http://www.bbraun.com.au)

B. Braun New Zealand | Level 8, 139 Quay Street, Auckland, 1010, New Zealand | Customer Care 0800 227 286 | Fax (09) 373 5601 | [www.bbraun.co.nz](http://www.bbraun.co.nz)