


Barrier Cream and Film Range

Lasting Skin Protection For
Stoma & Wound Care



Did you know

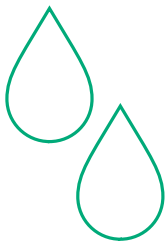

Moisture Associated Skin Damage (MASD) causes swelling & disruption of the stratum corneum¹

Prolonged exposure to various sources of moisture cause the skin to become hyperhydrated, macerated or excoriated, increasing the susceptibility to physical damage from friction, shearing forces and adhesive injuries.

ALLY® Barrier Film

No sting, rapid drying, breathable skin barrier

ALLY® Barrier Film is a rapid drying, transparent, breathable skin barrier that brings 48 to 72 hours protection to intact or damaged skin.


Helps **prevent** skin breakdown caused by moisture:

- Protection of vulnerable areas or sensitive, fragile skin: heels, elbows, toes and hips.
- Protection of incontinent patients' skin.

Helps **protect** skin at risk from maceration and excoriation:

- Periwound skin
- Peristomal area
- Around fixation sites for drainage tubes and external catheters

Helps **maintain** the integrity of newly healed skin.

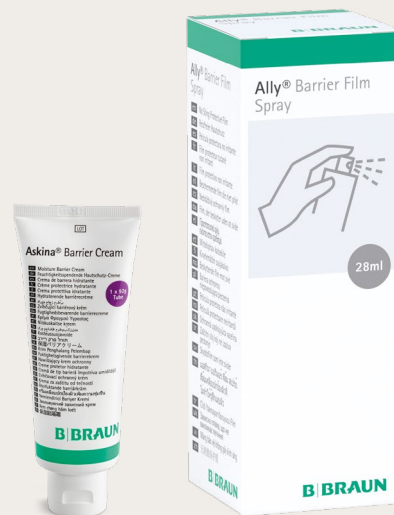

Benefits²:

- Completely sting free, can be used on damaged skin
- Rapid drying < 30 secs
- Transparent: allows visual control
- Provides 48-72 hours protection
- Easy to remove
- Offers protection to the skin against subsequent moisture contact
- The barrier spray forms a thin layer, repelling moisture and potential irritants, thus protecting the skin and keeping it dry so that stoma appliances and dressings remain intact

Barrier Film Range

Care pathway protocol for the protection of the skin

Clinical Presentation	What To Use	When To Use	How Much To Use
Peri-Wound Protection 		At each dressing change	Apply an even coat. Spray 10–15cm away from the surface. Allow 30 seconds to dry.
Peri-Stomal Protection 		At each pouch/application change	Apply an even coat. Spray 10–15cm away from the surface. Allow 30 seconds to dry.
Skin Fold Protection 		Once per day	Apply an even coat. Spray 10–15cm away from the surface. Allow 30 seconds to dry.
Medical Adhesive Related Skin Injury (MARS) Prevention 		At each dressing change	Apply an even coat. Spray 10–15cm away from the surface. Allow 30 seconds to dry.
Peri-Tube & Device Skin Protection 		At each dressing/tape change	Apply an even coat. Spray 10–15cm away from the surface. Allow 30 seconds to dry.
Skin Protection From Friction 		Once per day	Apply an even coat. Spray 10–15cm away from the surface. Allow 30 seconds to dry.
Incontinence Associated Dermatitis 		Apply Askina Barrier Cream day and night for low risk patients For high risk patients, use Ally Barrier Film Spray or Askina Barrier Swab twice a week	Askina Barrier Cream: Apply an even coat Ally Barrier Spray: Apply an even coat. Spray 10–15cm away from the surface. Allow 30 seconds to dry.

* Askina Barrier Cream should not be used on infected skin or where there is known allergic sensitivity to the ingredients.

Protecting from head to toe

For wound and stoma care

Friction from respiratory devices

Skin Stripping from adhesives -
central line dressings

Skin protection during radiation therapy

Friction over bony prominences


Moisture friction in skin folds

Incontinence associated dermatitis

Protection from moisture

Peri-wound maceration

Protection against pressure injuries
around bony prominences

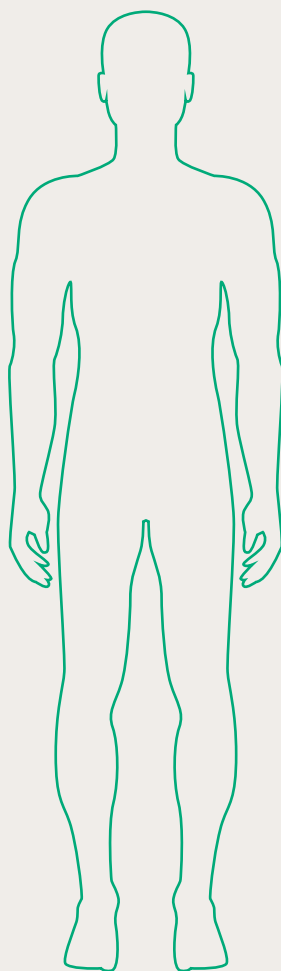

Skin tears from tapes used to
secure critical tubing

Moisture damage around mouth or
tracheostomy

Moisture damage around leaking tubes
or drains

Skin damage around surgical
incisions and fistulas

Damage from bodily fluids and
adhesives around ostomies

Skin stripping from adhesives -
peripheral catheters

Skin stripping from adhesives, tapes
and dressings

Skin tears

Friction over heels

Barrier Film Range

Size	Units/Box	Product Code
28 ml Spray Bottle	12	5036
1 ml Foam Swab	25	5037
92 g Barrier Cream	24	5038



B. Braun Australia Pty Ltd | Norwest Business Park NSW, 2153
Tel. 1800 251 705 | info.au@bbraun.com | bbraun.com.au

B. Braun New Zealand Pty Ltd | Level 8, 139 Quay Street, Auckland, 1010
Tel. 0800 227 286 | info.nz@bbraun.com | bbraun.co.nz