




Prontosan® Wound Gel X

Prevents the formation of biofilm

Prontosan® Wound Gel X

While necrotic tissue and slough can be optimally removed by Prontosan® Wound Irrigation Solution, it is also necessary to remove heavy encrustations by keeping the wound moist for a longer period of time. This can be achieved by simply applying Prontosan® Wound Gel X with a secondary dressing.

Prontosan® Wound Gel X is applied between dressing changes to continue wound cleansing and debridement, after the use of Prontosan® Wound Solution.

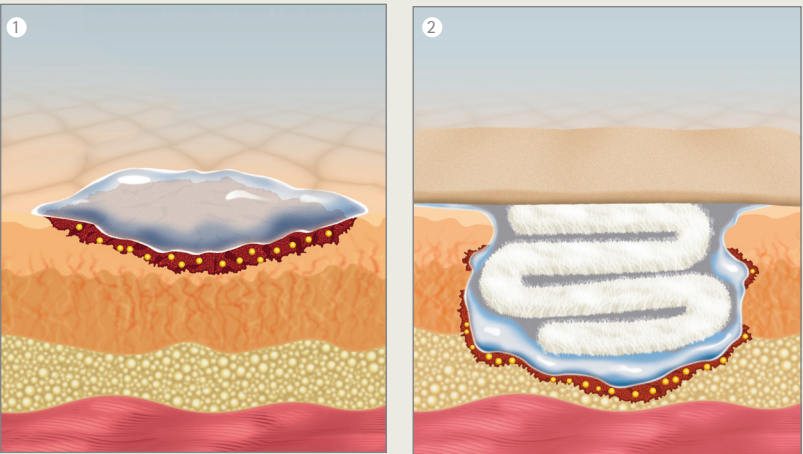

Thick gel for easy application



Indications

- Traumatic wounds
- Postoperative wounds
- Chronic wounds
- Thermal wounds
- Chemical wounds
- Radiation-induced wounds
- Moistening of dressings and bandages

Application

- Apply a 3–4 mm layer to large surface wounds and cover with a secondary dressing
- Apply a 3–5 mm layer to deep or difficult-to-access wounds and cover with a secondary dressing

Benefits

1. Removes & prevents biofilm¹
2. Prevents infections¹
3. Supports healing¹
4. Painless dressing changes¹
5. Low allergenic potential
6. Compatible with all secondary wound dressings*
7. Usable up to 8 weeks after opening²

*Excludes maggot dressings

When to use:

Prontosan® Wound Gel X	Prontosan® Wound Gel
50g tube	30 ml bottle
Highly Viscous	Fluid
For large surface wounds	For deep & tunneling wounds
For ulcers & pressure injuries	For hard-to-reach wounds
Burns (I, II & III degree)	Burns (I & II degree)
When larger quantities are needed	When smaller quantities are needed



B. Braun Ordering Information

Prontosan® Wound Gel X

Description	Size (cm x cm)	Pcs/Pack	REF
Prontosan® Wound Gel X	50g	20	400523
Prontosan® Wound Gel	30ml	20	400510

Reference:

- 1. Eberlein T, Assadian O. Clinical use of polihexanide on acute and chronic wounds for antisepsis and decontamination. Skin Pharmacol Physiol 2010;23 (Suppl 1):45-51
- 2. Horrocks A, (2006) Prontosan Wound Irrigation and Gel: Management of chronic wounds

B. Braun Australia Pty Ltd | Norwest Business Park NSW, 2153 | Tel. 1800 251 705 | info.au@bbraun.com | bbraun.com.au
B. Braun New Zealand Pty Ltd | Level 8, 139 Quay Street, Auckland, 1010 | Tel. 0800 227 286 | info.nz@bbraun.com | bbraun.co.nz

OM E0079.C 06/26 | ©2026 B. Braun Australia Pty Ltd. ABN 56 002 945 155. All rights reserved. In Australia, Prontosan® is a registered trade mark of B. Braun Melsungen AG. This brochure is intended for use by healthcare professionals only. Subject to technical changes. It is provided for informational purposes and does not constitute medical advice. Always refer to the product's Instructions for Use and consult a qualified healthcare provider for clinical decisions. This material may not be copied, reproduced, or distributed without express written authority of B. Braun Australia Pty Ltd.