


Ecoflac™ Mix – Transfer Cap

Convenient, needle-free and time-saving drug preparation



Transfer cap for transferring fluids between an Ecoflac® plus container and vials (cap diameter: 20 mm).

1

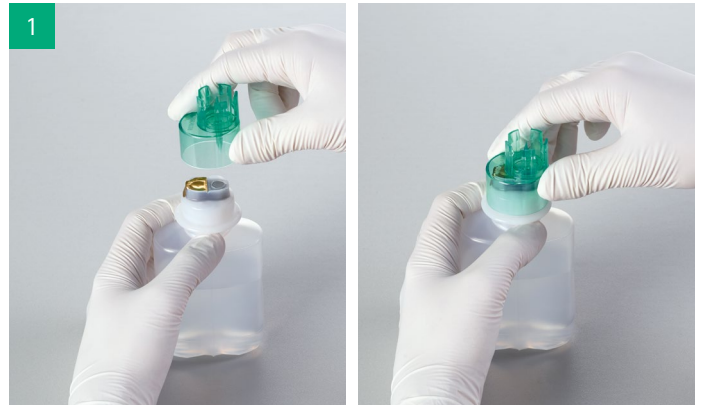

Place Ecoflac Mix on the upright standing container and puncture the port. No need to rotate the spike.

2



Attach the vial to Ecoflac Mix.

3



Turn the container upside down. By pressing the Ecoflac plus, pump some solution into the vial. Carefully shake until the lyophilisate drug is completely dissolved.

4



Then turn the container upright and pump the vial contents back into Ecoflac plus. Repeat these steps until the drug lyophilisate is completely transferred into Ecoflac plus.

5



Finally, remove Ecoflac Mix together with the attached vial to allow the insertion of the IV-set into Ecoflac plus.

Ecoflac™ Mix – Transfer Cap



Product benefits

- Provides a stable connection between both containers
- Specially designed to fit with Ecoflac plus and standard vials
- Prevents all risks related to the use of needles during reconstitution
- Ergonomic handling increases efficiency of healthcare workers
- Convenient handling helps to improve aseptic technique
- CO₂ emissions per item reduced by approx. 22.8%¹ in comparison between old and new design
- Weight reduction of new Ecoflac Mix compared to old version 13.7%¹



22.8% CO₂ savings per item¹



Not manufactured with Latex



Not manufactured with PVC



Not manufactured with DEHP and other Phthalates



Not manufactured with BPA

Ecoflac® Mix – transfer cap	Units Per Box	Code No. (REF)
	250 pcs.	16401N

Literature: 1. | Test Report – Life Cycle Assessment (LCA) Ecoflac Mix in accordance with ISO 14040/14044, 2024.

B. Braun Australia Pty Ltd | Level 2, 4 Burbank Place, Norwest NSW 2153, Australia | Tel.1800 251 705 | info.au@bbraun.com | www.bbraun.com.au
B. Braun New Zealand | Level 8, 139 Quay Street, Auckland, 1010, New Zealand | Customer Care 0800 227 286 | Fax (09) 373 5601 | www.bbraun.co.nz

E1178 2024 B. Braun Australia ©2025 B. Braun Australia Pty Ltd. ABN 56 002 945 155. All rights reserved. Ecoflac is a trademark of B. Braun Germany. This brochure is intended for use by healthcare professionals only. It is provided for informational purposes and does not constitute medical advice. Always refer to the product's Instructions for Use and consult a qualified healthcare provider for clinical decisions.