

Ally® Barrier Film Spray

Lasting skin protection for stoma and wound care



When to use Barrier Film Products?



- 1 TO PREVENT** skin breakdown caused by FRICTION or MOISTURE
 - Protection of vulnerable areas: heels, elbows, toes, hips – from the first signs of reddening caused by friction.
 - Protection of incontinent patients' skin.
- 2 TO PROTECT** skin at risk from hyperhydration and/or MACERATION and EXCORIATION
 - Periwound skin.
 - Peristomal area.
 - Around fixation sites for drainage tubes and external catheters.
- 3 TO MAINTAIN** the integrity of newly healed skin to stop secondary breakdown



Skin at risk:
incontinent patient



Incontinence dermatitis
with skin breakdown



Skin at risk:
maceration




Skin at risk:
Tube insertion site

Indications

Ally® Barrier Film Spray and Askina® Barrier Film swabs is indicated for use on intact or damaged skin. The Ally® Barrier Film Spray acts as a protective barrier:

- Against irritation from bodily fluids
- For sensitive and fragile skin
- Under adhesive dressings on peri wound area to reduce disruption to newly healing tissue
- For damaged skin
- For skin tears with skin flap intact or category 1a, 1b, 2a, 2b
- To protect periwound and peristomal areas
- For small cuts and tears
- At fixation sites for drainage tubes and external catheters

Completely sting free

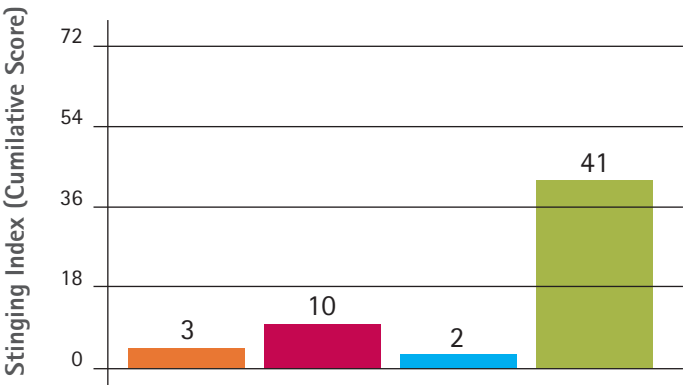

■ Ally® Barrier Film Spray
■ Market leading barrier film
■ Saline solution
■ Alcohol

Results of a study*:
 19 patients rated stinging on a scale of 0-4
 0 (no stinging), 1 (mild), 2 (moderate),
 3 (severe), 4 (extreme)

* Data on file

How to use Ally® Barrier Film Spray?



1 Skin preparation

- Cleanse the skin with appropriate cleansing solution and dry thoroughly.

2 Application

- Holding the spray nozzle 10 - 15 cm from the skin, press the spray nozzle and apply a smooth coating of film over the necessary area.
- Allows the film to dry for 30 seconds.
 - If using the Askina Barrier Film swab, apply to area requiring protection and
 - Allow 30 seconds to dry.


3 Removal

- If necessary, can be removed with Ally Adhesive Remover or a commonly used medical adhesive remover.

Benefits



Day 1 Significant irritation of peristomal skin at day 1.

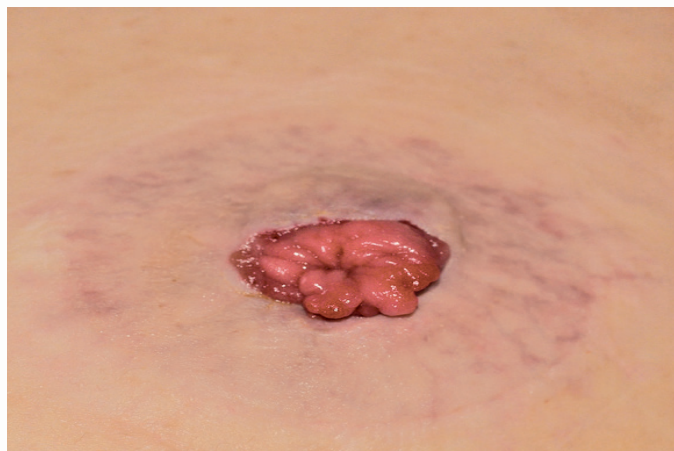


Day 15 Signs of irritation have been almost entirely resolved at day 15.

Patient's main benefit in stomacare*:
Secure adhesion of pouch
Less pain during pouch removal

*Data on file.

**User opinion
on skin protection efficacy¹**

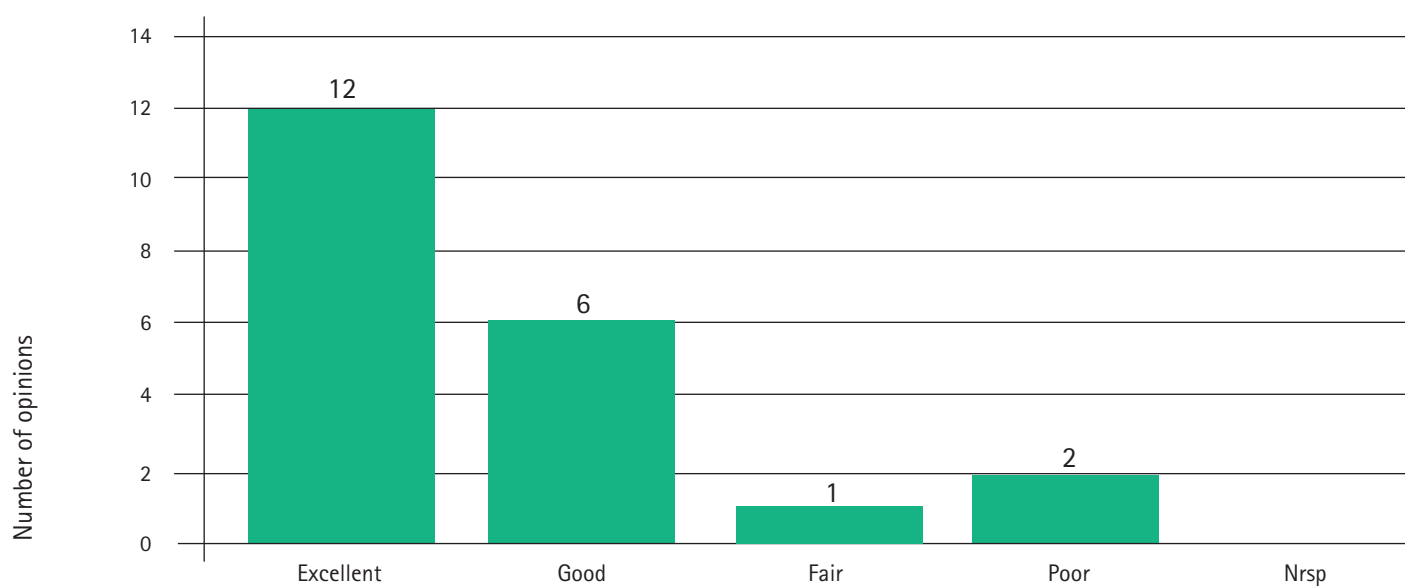

1. 21 patients took part in the survey* In summary, 18 out of 21 patients rated the product as «Excellent» or «Good». Data on file.

- **How often can Ally® Barrier Film Spray and Askina® Barrier Film Swabs be applied?**
The protective film will last between 48 and 72 hours, but re-application is recommended each time you change your stoma pouch. For IAD if you can still see the sheen, then there is no need to re-apply.
Guidelines recommend to re-apply after every 3 washes.
- **How should I remove Ally® Barrier Film Spray and Askina® Barrier Film Swabs?**
The barrier film products can be removed using Ally® Adhesive Remover or a commonly used medical adhesive remover.
- **Can Ally® Barrier Film Spray and Askina® Barrier Film Swabs be used on damaged or sensitive skin?**
Yes, it contains no alcohol, and can be safely used to provide pain-free protection on damaged, sensitive and fragile skin.
- **Can Ally® Barrier Film Spray and Askina® Barrier Film Swabs be used on infants?**
The Barrier film products can be used on infants over 1 month of age.
- **How long does the bottle last?**
On average, Ally® Barrier Film 28 ml spray will last for 140 pumps, which equates to one months usage.
- **Will Ally® Barrier Film Spray and Askina® Barrier Film Swabs dry my skin?**
Biological tests have been carried out on Ally® Barrier Film Spray and Askina® Barrier Film Swabs and there is scientific evidence that products will not dry your skin. When applied to the skin it allows the skin to breath normally.
- **If my peristomal skin itches under my ostomy devices, could I use Ally® Barrier Film Spray and Askina® Barrier Film Swabs?**
Peristomal skin is damaged by frequent removal of ostomy devices, which can cause irritation. The use of these products will protect your skin and prevent any damage, therefore prevent irritation and itching.

B. Braun Ordering Information

Ally® Barrier Film Spray and Askina® Barrier Film Swabs

Size	Pcs/Pack	REF
28 ml Spray Bottle	10	5036
1ml Foam Swab	25	5037

B. Braun Australia Pty Ltd | Norwest Business Park NSW, 2153 Tel. 1800 251 705 | info.au@bbraun.com | bbraun.com.au

B. Braun New Zealand Pty Ltd | Level 8, 139 Quay Street, Auckland, 1010 Tel. 0800 227 286 | info.nz@bbraun.com | bbraun.co.nz