



Prontosan® and Askina® Calgitrol™

Prevention and treatment of wound
infection

Prontosan® – wound bed preparation

The wound bed preparation system that helps cleanse, debride, and prevent infections to enable better therapeutic cleansing outcomes.

Prontosan® Wound Irrigation & Gel

Indications

Prontosan® Wound Irrigation Solution and **Prontosan® Wound Gel / Gel X** are indicated for cleansing and moistening of acute, chronic, infected thermal and non-thermal burns IIa, IIb, and III, intraoperative cleansing and irrigation of wounds, postoperative wounds, fistulas and abscesses, and peristomal skin.

Prontosan® Wound Irrigation Solution is also ideal for moistening encrusted dressings, or bandages prior to removal and for instillation in combination with negative pressure wound therapy.

Prontosan® Gel and Gel X act as an effective barrier to reduce microbial penetration through the dressing and to decontaminate the wound bed between dressing changes.




Advantages

- Management and prevention of biofilm reformation
- Helps to prevent infections
- Improved patient outcomes, including time to heal
- Well-known substances with low allergenic potential
- Can be used up to 8 weeks after first opening

Prontosan® Debridement Pad

Indications

Prontosan® Debridement Pad has been designed to support the Wound Bed Preparation with Prontosan® Wound Irrigation Solution. **Prontosan® Debridement Pad** frees the wound from coatings and dead cell residues (debris) and absorbs excess exudates and slough. Intact tissue is spared. **Prontosan® Debridement Pad** produces good results even with scaly and necrotic coatings, if they are subject to prior autolytic debridement.

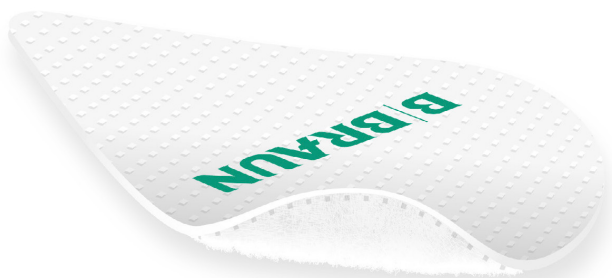

Prontosan® Debridement Pad is intended for single use only

Advantages

- Good cleansing and debridement due to microfiber technology
- Soft debridement, no tissue irritation
- Unique droplet shape to allow debridement of cavities and areas difficult to reach
- Blister packaging to allow safe and aseptic soaking of the pad prior to use
- Cleansing sheet composed of polyester and polypropylene microfiber supported by a backing layer made from polypropylene.

Askina® Calgitrol™ – Broad antimicrobial effectiveness

Treatment of local infection is essential for the healing process

Askina® Calgitrol™ Paste is a sterile primary dressing consisting of an ionic silver alginate matrix in paste form, which provides antimicrobial barrier and helps prevent contamination from external bacteria. In the presence of wound exudate, the silver alginate matrix helps maintain a moist wound environment conducive to natural healing conditions. The high conformability allows a close contact to the wound bed.

Indications

Askina® Calgitrol™ Paste are indicated for the management of exuding, partial to full thickness wounds, stage I-IV pressure injuries, venous ulcers, second degree burns and donor sites.

Askina® Calgitrol™ Paste is ideally suited for the management of tunnel wounds and small sinuses like in diabetic foot.


Advantages

- Broad antimicrobial effectiveness
- Immediate availability of silver ions
- Sustained controlled release to the wound bed during use of the dressing
- Tolerable
- Easy to use, conformable



Askina® Calgitrol™ Paste
Amorphous silver alginate matrix

Patented ionic silver alginate matrix –
Mode of actions:

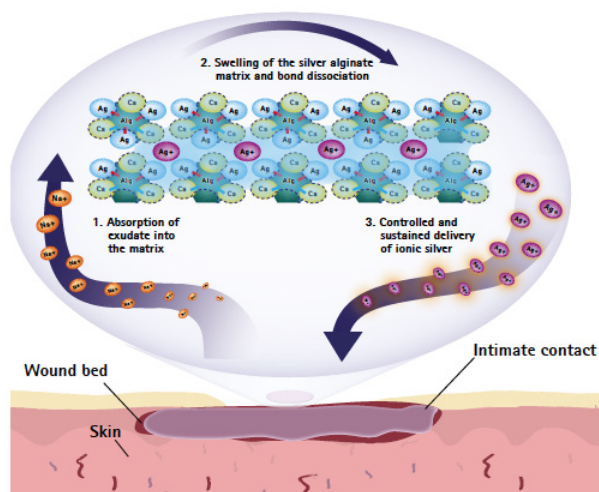

Askina® Calgitrol™ Paste



Askina® Calgitrol™ Paste is conformable and allows an extremely close contact between the active ionic silver alginate matrix and the wound bed, which is particularly valuable in difficult to manage wounds such as tunnels and sinuses. Cover it with an appropriate secondary dressing depending of the amount of exudate

Ordering Information

Prontosan® Product Details



Product	Pack size	Product Code
Prontosan® Irrigation Solution		
40ml ampoule	30	400419
350ml bottle	10	400431
1000ml bottle	10	400432
75ml spray bottle	20	400565



Product	Pack Size	Product Code
Prontosan® Wound Gel - 30ml	20	400510
Prontosan® Wound Gel X - 50gm	20	400523



Product	Pack Size	Product Code
Prontosan® NPWT Instillation Adapter to be used with the Prontosan® Irrigation Solution 1000ml bottle (compatible with V.A.C. Veraflo).	10	3908437



Product	Pack Size	Product Code
Prontosan® Debridement Pad	10	3908457
Prontosan® Debridement Pad	3	3908456



Product	Pack Size	Product Code
Askina Calgitrol™ Paste 15g	10	7211599

*Reference: Data on File

B. Braun Australia Pty Ltd | Norwest Business Park NSW, 2153
Tel. 1800 251 705 | info.au@bbraun.com | bbraun.com.au

B. Braun New Zealand | PO Box 37353, Parnell, Auckland 1151
Tel. 0800 227 286 | info.nz@bbraun.com | bbraun.co.nz